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#### Popis výrobku

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	Contents remain sterile unless peel packs are damaged.		
	Do not re-sterilize.		
	Keep out of reach of children.		Date of revision of the text: 2016-09

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## EN Instructions for use

### Product description

HydroClean cavity is a hydroactive wound dressing that contains, as a core component, a superabsorbent polyacrylate (SAP) embedded in cellulose fibres and activated (premoistened) with Ringer's Solution. The wound pad is covered with polypropylene fabric on all sides.

### Properties and mode of action

HydroClean cavity delivers Ringer's solution to the wound for up to three days. During this time, interactive and continuous wound irrigation takes place. Due to the wound dressing pad also absorbing wound exudate. The SAP used in HydroClean cavity inactivates matrix metalloproteinases (MMP) which impair wound healing. As a result, stagnating healing in chronic wounds can be reactivated.

### Indications

HydroClean cavity is suitable for wetmoist treatment of wounds, in particular of wounds with impaired healing tendency. The product is especially suitable for the treatment of chronic and poorly healing wounds during the cleansing and granulation stage. HydroClean cavity can also be used for the treatment of infected wounds. HydroClean cavity can furthermore be used to reactivate stagnating healing processes if these are caused by excessive MMP activity.

### Mode of application

Pack the wound with HydroClean cavity and cover with a HydroClean Wundkissen abgedeckt. HydroClean cavity must be in direct contact with the wound bed. The upper HydroClean wound dressing pad should overlap the wound edges and should be secured with suitable dressing retention material. If the condition of a wound when packed with HydroClean cavity does not allow to be covered with a HydroClean dressing, then a film dressing can be used instead. The dressing can remain in situ for up to three days, dependent on clinical judgement. If the wound requires large amounts of moisture, the wound dressing may dry out prematurely and may require an earlier dressing change. In most cases, HydroClean cavity can be removed from the wound without any problems. Should the wound dressing pad adhere to the wound, it can be irrigated with physiological saline solution or Ringer's solution prior to removal and can then be removed almost painlessly after a few minutes.

### Side effects

At the start of treatment with HydroClean cavity, breakdown of irreversibly damaged tissue can lead to an increase in the size of the wound. This can be a sign that wound healing has commenced. Reddening of the blood edges can occur during wetmoist treatment; this is normally a sign that blood circulation has been reactivated as the wound begins to heal. However, the cause must be assessed individually in each case. HydroClean cavity therapy itself does not cause any pain. However, an open wound is often highly sensitive to mechanical stimuli. The question whether an adequate analgesic therapy is indicated must be considered on a case-by-case basis. Whilst wetmoist treatment with Ringer's solution does not lead to maceration of vital cells, it can cause swelling of the soaked plantar horn layer, which then appears as a whitish coating.

### Contraindications

Do not use HydroClean cavity on patients with an intolerance to any of its components.

**Special precautions**
HydroClean cavity must not be cut to size or otherwise damaged mechanically. If the skin around the wound is particularly sensitive and impaired, it is advisable to protect the skin with suitable skin protection. In patients with a risk of excessive bleeding (e.g. due to blood-thinning medication), HydroClean should be removed with caution at each dressing change necessary. HydroClean should first be irrigated with Ringer's solution or physiological saline solution as described above. Infected wounds and wounds highly exposed to bacteria should only be treated under the supervision of a physician or other healthcare professional, particularly if a film dressing is used for fixing in place.

**CAUTION:** Before treating wounds with impaired healing tendency, the wound condition and the causes of the impairment must always be assessed by a physician or other healthcare professional. Treatment with HydroClean cavity cannot replace treatment of the cause of impaired wound healing.

### Special instructions

• Store in a dry place.

• Contents remain sterile unless peel packs are damaged.

• Do not re-sterilize.

Keep out of reach of children. Date of revision of the text: 2016-09

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