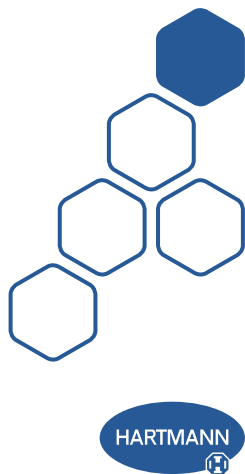
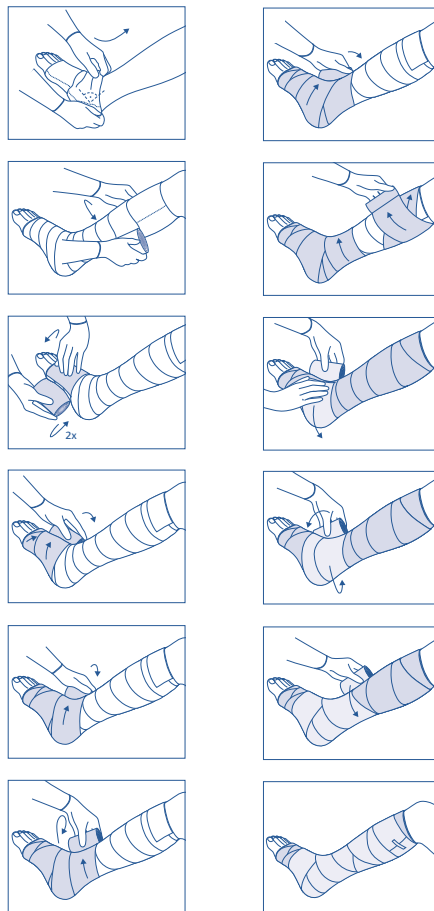
Pütter® Flex

Bi-elastische Kurzzugbinden
Bi-elastic short-stretch bandages
Bandes bi-extensibles à allongement court
Bi-elastische zwachtels met korte rek
Bende bielastiche a corta trazione
Vendas bielásticas de corta elasticidad
Ligaduras bielásticas de curta tração
Επίδεσμοι μικρής τάνυσης με οριζόντια και κάθετη ελαστικότητα
Opaski bi-elastyczne o niskiej rozciągłości
Бинты короткой растяжимости, способные растягиваться в обоих направлениях
Bandaje bi-elastice cu elasticitate redusă
Bielastiska kortsträcksbindor
Bielastiset vähäelastiset siteet
雙彈性短拉力繃帶
أربطة مزدوجة المرونة قصيرة المدى

1

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EN Instructions for use

General description of the device

PütterFlex® is a brown-coloured, bi-elastic short-stretch bandage with textile elasticity and an extensibility of approximately 90 % lengthwise and approximately 40 % widthwise for very high compression.

Intended purpose

PütterFlex can be used for long-term compression therapy:

- treatment of chronic venous insufficiency (CVI) with and without venous leg ulcers (VLU)
- thrombosis / thrombophlebitis
- chronic oedemata of other origins (ankle brachial pressure index – ABPI 0.8–1.3)
- lymph oedemata

The product can be used on humans by healthcare professionals in clinical and home environment in combination with padding bandages and inner lining material. PütterFlex is reusable.

Properties

Extensibility: approx. 90% lengthwise, approx. 40% widthwise.

Washable: Wash the bandage in a regular wash cycle at a temperature of 95°C.

Indication

The product can be used for the treatment of varicose veins, thrombosis / thrombophlebitis, chronic venous insufficiency (CVI), primary and secondary prevention of leg ulcer venous/venous insufficiency and oedemata of other origins (ankle brachial pressure index - ABPI 0.8 - 1.3).

Intended users

PütterFlex is designed to be used by healthcare professionals who have received appropriate training. PütterFlex can be used for the treatment of: varicose veins, thrombosis / thrombophlebitis, post-thrombotic syndrome, thromboprophylaxis, chronic venous insufficiency (CVI), venous insufficiency, angiodysplasia and other oedemata.

Composition

100 % cotton

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DE – PAUL HARTMANN AG · Paul-Hartmann-Straße 12 · 89522 HEIDENHEIM, GERMANY
CH – IVF HARTMANN AG · 8212 Neuhausen