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Produktbeschreibung

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Produkt description

Zetivut Plus Silicone is a sterile dressing for the treatment of moderately to highly exuding, acute and chronic wounds. The silicone layer in contact with the wound allows easy application and almost painless, atraumatic removal. The absorbent pad absorbs and retains the exudate.

Composition

Zetivut Plus Silicone comprises an absorbent pad made from cellulose and superabsorbent polyacrylate for the absorption and retention of exudate. On the wound side it is covered with a silicone layer to facilitate application and atraumatic removal. The dressing consists of a mild, acetic acid containing polymer, which allows the dressing to adhere to the wound. The backing of the dressing is made of green-coloured, polypropylene nonwoven, which is hydrophobic but permeable to air and allows for gas exchange.

Properties and mode of action

Zetivut Plus Silicone absorbs exudate and retains it in the absorbent pad. The micro-absorbent silicone contact surface prevents the dressing from slipping and secures its application. The dressing changes are carried out atraumatically and almost painlessly.

Intended purpose

Zetivut Plus Silicone is a single use sterile superabsorbent dressing with a silicone interface for long-term treatment of injured skin, acute and chronic wounds, with

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Special precautions

The wound dressing may not be cut to size. Before treating a wound with healing impairment, a clinician has to be consulted to establish a clinical diagnosis. Treatment with Zetivut Plus Silicone cannot replace a causal treatment of the wound-healing impairment. If there are clinical signs of infection, the infection needs to be controlled with appropriate treatment before this dressing can be used.

In all cases, follow the established clinical protocol.

In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressing should be used with caution and under medical supervision.

- It is not recommended to stand on the dressing, e.g. in the case of wounds on the feet.

- For the adhesive strips of the adhesive bandage, the adhesive part should be attached to the dry and protected area of the skin.

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

Side effects

Increased exudate, maceration, increased pain, adherence of the dressing to the wound (itching, burning, oedema, multiple ulceration, skin torn, blisters, erythema). If any of these side effects occur, consult a clinician.

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