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**Intended purpose**  
 ResporSorb Silicone is a single use sterile superabsorbent dressing with a silicone interface for long-term treatment of injured skin acute and chronic wounds, with moderate to high levels of exudate. It is used on adults only, by healthcare professionals in clinical or healthcare settings and can be combined with local antiseptics, primary and secondary dressings and compression bandages.

**Properties and mode of action**  
 ResporSorb Silicone absorbs exudate and retains it in the absorbent pad. The micro-adhesive, silicone interface prevents the dressing from slipping and facilitates its application. The dressing changes are carried out atraumatically and almost painlessly.

**Indications**  
 ResporSorb Silicone is suitable for the treatment of wounds with moderate to high levels of exudate. Under other words: acute wounds (skin diseases, burns, post-operative wounds) and chronic wounds (ulcers, pressure sores, tumor wounds, diabetic feet). The dressing is used only on adults and only in clinical or healthcare settings and can be combined with local antiseptics, primary and secondary dressings and compression bandages.

## Indicazioni

ResporSorb Silicone is suitable for the treatment of wounds with moderate to high levels of exudate. Under other words: acute wounds (skin diseases, burns, post-operative wounds) and chronic wounds (ulcers, pressure sores, tumor wounds, diabetic feet).

## Mode of application

- Clean the wound using a physiological solution or as instructed by a clinician.
- Select a suitable dressing size for the wound making sure that the dressing overlaps the margins of the wound by approx. 1–2 cm. Do not cut the dressing.
- Before applying the dressing, after rinsing the wound, make sure that the area around the wound is dry using swabs to ensure that the dressing adheres.
- Remove the protective sheets. Place the dressing with the adhesive side onto the wound, so that the green face stays away from the wound.
- Secure the dressing using e.g. PeHa-haft bandage or any other suitable adhesive.
- If need be, with compression bandages as instructed by a clinician.
- A dressing can remain on the wound for up to 5 days, depending on the condition of the wound and the established clinical protocol. The dressing must be changed if clinical or healthcare reasons dictate or when exudate reaches the rim of the dressing.

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## Contraindications

ResporSorb Silicone in case of hypersensitivity to any of its components.

## Special precautions

- The wound dressing may not be cut to size.
- Before treating a wound with healing impairment, a clinician has to be consulted to establish a clinical diagnosis. Treatment with ResporSorb Silicone cannot replace a causal treatment of the wound-healing impairment. If there are clinical signs of infection, the infection needs to be controlled with appropriate treatment before this dressing can be used.
- In all cases, follow the established clinical protocol.

In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, in these population groups this dressing should be used with caution and following a clinician's recommendation.

- It is not recommended to stand on the dressing, e.g. in the case of wounds on the feet.

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