HARTMANN
Peha-instrument
Irrigating cannula 18 G L 81 mm
(Ref 9910114)

EN
INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION:

The Irrigating cannula 18 G L 81 mm is a single-use cannula made out of steel with a very clean lumen. It is available only in one diameter and length. The cannula hub is fitted with a Luer-Lock, that prevents disconnection during use. The hub is transparent to see fluid flow. Product is sterilized using Ethylene oxide and intended for transient use (<60 minutes).

INTENDED USE
The Irrigating cannula is used for the washing and irrigating of wounds.

TARGET POPULATION
For human use – no age restrictions

INTENDED USERS
Health care professionals only

CONTRAINDICATIONS AND SIDE-EFFECTS
Product is not expected to have any contraindications or present any side-effects

INSTRUCTION FOR USE

- Check expiry date prior to use
- Inspect the integrity of the primary packaging before use. Never use the device when the primary packaging is damaged.
- Open the cannula’s packaging and set out its content on a sterile operation area. During intervention, respect strict aseptic good practices.
- Protective glasses could be worn during the use
- Carefully check if the olive is attached to the cannula before use.
- Use the product immediately after opening the primary packaging.
- Connect the irrigating cannula to a 30-35cc syringe and prepare 100ml of cleansing solution:
- Start at one edge of the wound
- Hold the end of the irrigation cannula 10-15cm (4-6 inches) from the wound.
- Angle the irrigation cannula towards the wound bed
- Apply full force on the syringe plunger and slowly ‘sweep’ across the wound bed; refill the syringe as needed.
- Repeat the process from the opposite side of the wound.
- After using the cannula needle, check if the olive is still attached to the cannula.

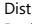
- WARNINGS:**
- Irrigating cannulas should be handled and operated by healthcare professionals completely familiar with their use, assembly, and disassembly.
 - Use instruments for their intended surgical purposes only.
 - Irrigating cannulas are to be used in accordance with these instructions for use. Read all sections of this insert prior to use. Improper use of the instrument may cause a serious injury.
 - At no time, twist or bend the cannula, or use the device as a scraping material against tissue or bones.
 - Make sure other mechanical devices do not come into contact with the cannula.
 - Do not use in blood vessels

In the case where the product is defective, please follow these instructions:
Do not use the product.
Keep the defective cannula needle.
Inform Aesthetic Group about this incident with the following data: reference + designation of product + batch + description of problem.
Ship the defective product to Aesthetic Group.

- CAUTIONS:**
- Read carefully the instructions before use
 - Do not re-sterilize.
 - Do not use the cannula if one of its components is damaged or if a component is missing. In case of fall or degradation of one of the components, stop using the cannula.

The smaller the diameter of the syringe the greater the psi; pressures higher than 15 psi can damage the wound bed and may drive bacteria into the deeper tissues.

Distributed by: PAUL HARTMANN AG
Paul-Hartmann Strasse 12 – 89522 HEIDENHEIM, GERMANY

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