Vivano[®] Med Gel Strip

Gelstreifen
Gel strip
Bande gel
Gel in striscia
Gelstrip
Banda de gel
Tira de gel
Gel em tira
Επίθεμα γέλης
Gelový proužek
Gélová páska
Pasek želowy
Géltapas
Гелевая полоска
Гел лента
Ljepljiva gel-traka
Bandă cu gel
Trak z gelom
Jel Şerit
醫用膠帶
شریط چل

Vivano[®]
Safety. And Simplicity.

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English

Instructions for use

The Vivano Med Gel Strip may only be used by a physician or in accordance with medical instructions. The Vivano Med Gel Strip is intended for use in conjunction with the Vivano Tec negative pressure therapy system by PAUL HARTMANN AG.

Properties

Vivano Med Gel Strip is a hydrogel strip on the basis of glycerol/water/polysulfonate with a polyester fabric core. It is adhesive on both sides and can be cut to size.

Intended use

The Adhesive Hydrogel Strip may be used to assist with seal integrity and/ or securing fixtures in position at the wound site.

Indication for use

The Vivano Med Gel Strip is used to fix tubes and to efficiently seal applied wound dressings as part of Negative Pressure Wound Therapy (NPWT), also in anatomically difficult areas, such as in the sacral region.

Mode of application

1. Ensure that the skin on which the gel strip is to be applied is clean and dry.
2. Remove the sterile gel strip from the peel pouch.
3. If necessary, the gel strip can be cut to the required size with sterile scissors.
4. Remove the plastic film backing from both sides before applying the gel strip.
5. Apply the gel strip on the respective area of the skin.

Monitoring the wound

In order to guarantee that the therapy is safe, the wound dressing has to be checked frequently. The wound dressing must be checked for imperviousness and the wound margins as well as the exudate for signs of infection. If there are any signs of infection the attending physician must be notified immediately.

Dressing change

The dressing must be changed at regular intervals of 48 to 72 hours. In the case of infected wounds, the dressing should be changed at intervals of 12 to 24 hours.

Adhesion of the dressing to the wound

If the Vivano Med Gel Strip is difficult to remove, irrigate the product with a sterile saline solution or any other suitable sterile rinsing solution and then remove.

Contraindications

Contraindications are present in patients with malignant tumour wounds, open or exposed joints and untreated osteomyelitis, open peritoneal spaces, pleural cavity or non-enteric and unexamined fistulas, wounds covered with necrotic tissue, eschar or more than 30% slough. The Vivano Med Gel Strip must not be applied directly over exposed nerves, anastomosis points, organs, arteries or veins.

Do not use the Vivano Med Gel Strip on patients who may be allergic to any of its ingredients.

In case of an allergic reaction, remove the product immediately and clean the affected area.

Precautions and Warnings

The product is intended for single use only.

Do not use after expiry date.

For short term use only. Do not continue dressing regimen for more than 30 days.

Discontinue use if excessive discomfort or irritation develops.

All offcuts of the product must be discarded.

Using the product together with creams or lotions can reduce its adhesion.

Ensure that both plastic film backings have been removed before use.

Keep out of reach of children.

The CE mark indicates that this product meets the relevant requirements of the EU Directives

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HK – PAUL HARTMANN Asia-Pacific Ltd.