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Disiloxane, Trimethylsiloxysilicate, Trimethylsiloxylphenyl Dimethicone, Ethylhexyl Methoxycinnamate, Diethylamino Hydroxybenzoyl Hexyl Benzoate.

Stand der Information: 2017-01

What is the spray plaster and when is it used?

EN

The spray plaster forms an extremely thin, transparent, protective film and is used to cover abrasions, scratches and superficial cuts comfortably and easily. It protects the skin from dirt, bacteria and water while allowing it to breathe. It also contains ingredients for protection against UVA and UVB radiation.

The spray film does not contain alcohol and is odourless. It does not sting and forms an even protective layer over the skin. Especially in areas that are difficult to access, the spray film protects the skin discreetly and enables it to be inspected at all times. The spray plaster peels off the skin by itself and does not have to be removed.

What precautions should be taken when applying the spray plaster?

Before applying the spray plaster the skin must be washed very thoroughly. The skin surrounding the abrasions, scratches and superficial cuts must be clean and dry.

Apply the film to the skin evenly from a distance of approximately 10 to 15 cm. The spray plaster will dry within about 30 seconds. Apply a second layer if necessary.

When must the spray plaster not be used?

The spray plaster must not be used on wounds that are infected, bleeding or weeping. The film must not be sprayed onto mucous membranes or into the eyes. If this happens, the affected area should be washed with plenty of water.

Precautions when applying the spray plaster

Monitor the skin carefully if skin irritation or redness occurs. Should these symptoms worsen or persist, consult a doctor immediately.

Precautions

The liquid spray plaster is highly flammable. It should therefore be used only in well-ventilated areas. Avoid using near naked flames.

Keep out of the reach of children. Store at room temperature in order not to reduce the effectiveness of the spray plaster.

Ingredients:

Disiloxane, Trimethylsiloxysilicate, Trimethylsiloxylphenyl Dimethicone, Ethylhexyl Methoxycinnamate, Diethylamino Hydroxybenzoyl Hexyl Benzoate.

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PAUL HARTMANN AG
ES – Laboratorios HARTMANN S.A.
GB – PAUL HARTMANN Ltd.
IT – PAUL HARTMANN S.p.A.
NL – PAUL HARTMANN B.V.
BE – N.V. PAUL HARTMANN S.A.