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English

Congratulations on your purchase of this HARTMANN product. Please read the instructions carefully before using the thermometer for the first time, and keep them in a safe place. This product is intended for the measurement of human body temperature.

Explanation of symbols

➤: battery is empty

Err.: for errors, misreadings

Lo °C: temperature under 32 °C

Hi °C: temperature over 42.9 °C

Observation of the operating instructions

Protection against electric shock (type B)

Disposal note

Methods of measuring temperature

In the rectum (rectal)
This is by far the most accurate method from a medical point of view, because it comes closest to the core body temperature. The thermometer tip is inserted carefully into the rectum to a maximum of 2 cm. The usual measurement time is approximately 40 to 60 seconds.

Under the arm (axillary)

Placing the thermometer in the armpit provides a measurement of surface temperature that can fluctuate by around 0.5 °C to 1.5 °C from rectal temperature readings in adults. The usual measurement time for this method is approximately 90 seconds. It should be noted, however, that an exact reading cannot be obtained if, for example, the armpits have been allowed to cool. If this is the case, we recommend extending the measurement time to 5 minutes in order to obtain the most precise possible reading.

In the mouth (oral)

There are different heat zones in the mouth. As a general rule, the oral temperature is 0.3 °C to 0.8 °C lower than the rectal temperature. To ensure that the reading is as accurate as possible, place the thermometer tip under the tongue to the left or right of the root of the tongue. The sensor must have constant contact with the tissue during the reading and be placed under the tongue in one of the two heat pockets at the back. Keep the mouth closed during the reading and breathe evenly through the nose. Do not eat or drink anything before the measurement. The usual measurement time is approximately 60 seconds.

Note:
However, please note that Thermal standard is a maximum thermometer, i.e. the displayed temperature can increase slightly if measurement continues after the beeps. We strongly recommend the rectal method as the most accurate method for identifying the basal temperature, and advise you to extend the measuring time by 3 minutes after the beeps have sounded.

Cleaning and disinfection

The best way to clean the thermometer tip is by applying disinfectant (e.g. 70% ethyl alcohol) with a damp cloth.

Safety precautions

■ Do not allow the device to come into contact with hot water.

■ Do not expose to high temperatures or direct sunlight.

■ Do not touch the thermometer. It is neither shock-proof nor impact-resistant.

■ Do not bend or open the device (except the battery compartment).

Do not clean with thinners, petrol or benzene! Do not soak the thermometer in liquid. The thermometer contains small parts (battery, battery cover) that can be swallowed by children. Do not therefore leave the thermometer unattended when children are present. If the ambient temperature is above 35 °C immerse the thermometer tip in cold water for approximately 5 to 10 seconds before measuring the temperature.

A persistently high temperature, particularly in children, must be treated medically – please contact your doctor!

Battery replacement
The battery is empty and needs replacing when the symbol ➤ appears on the bottom right of the digital display. Remove the battery cover and replace it with a battery of the same type. Please note: the + sign on the battery must point downwards. Close the battery compartment, making sure that the catch clicks into place.

Product disposal
Please respect the environment. Thermometers and batteries should not be disposed of in household waste. Hand them into collection centres or municipal recycling centres for special waste.

Technical data
Type: Maximum thermometer
Measurement range: 32 °C to 42.9 °C
Accuracy: ±0.1 °C in the range 35.5 °C to 42 °C
±0.2 °C for the remaining LCD display, (display unit 0.1 °C)

Storage/transportation temperature: –25 °C to + 55 °C, relative humidity of ≤ 95 %

Ambient temperature during use: 10 °C to 35 °C, relative humidity of ≤ 80 %

Battery type: Alkaline-manganese battery, type LR 41, 1.55 V, battery life ~3,000 measurements

Weight: Approx. 11 g incl. battery

PAUL HARTMANN AG, Paul-Hartmann-Strabe, 89522 Heidenheim, Germany

Do not clean with thinners, petrol or benzene! Do not soak the thermometer in liquid. The thermometer contains small parts (battery, battery cover) that can be swallowed by children. Do not therefore leave the thermometer unattended when children are present. If the ambient temperature is above 35 °C immerse the thermometer tip in cold water for approximately 5 to 10 seconds before measuring the temperature.

A persistently high temperature, particularly in children, must be treated medically – please contact your doctor!

Battery replacement
The battery is empty and needs replacing when the symbol ➤ appears on the bottom right of the digital display. Remove the battery cover and replace it with a battery of the same type. Please note: the + sign on the battery must point downwards. Close the battery compartment, making sure that the catch clicks into place.

Product disposal
Please respect the environment. Thermometers and batteries should not be disposed of in household waste. Hand them into collection centres or municipal recycling centres for special waste.

Technical data
Type: Maximum thermometer
Measurement range: 32 °C to 42.9 °C
Accuracy: ±0.1 °C in the range 35.5 °C to 42 °C
±0.2 °C for the remaining LCD display, (display unit 0.1 °C)

Storage/transportation temperature: –25 °C to + 55 °C, relative humidity of ≤ 95 %

Ambient temperature during use: 10 °C to 35 °C, relative humidity of ≤ 80 %

Battery type: Alkaline-manganese battery, type LR 41, 1.55 V, battery life ~3,000 measurements

Weight: Approx. 11 g incl. battery

PAUL HARTMANN AG, Paul-Hartmann-Strabe, 89522 Heidenheim, Germany

This high quality product is covered by a 2-year warranty from the date of purchase, subject to the conditions listed below. Claims must be presented with proof of purchase within the warranty period. This product has been manufactured with the greatest care in accordance with international quality guidelines. Should you still have cause for complaint, please send the instrument, together with proof of purchase, to the service address below. Batteries are excluded from the warranty. The warranty does not cover damage resulting from improper use or unauthorised operation. Claims for compensation are limited to the value of the merchandise; compensation for subsequent damage or injury is not covered.

AE – PAUL HARTMANN Middle East FZE Dubai

AU – PAUL HARTMANN Pty. Ltd. Level 6, 5 Ryder Boulevard Rhodes, NSW 2138 Australia

GB – PAUL HARTMANN Ltd. Heywood/Lancashire OL10 2TT

HK – PAUL HARTMANN Asia-Pacific Ltd. Hong Kong

ZA – HARTMANN South Africa 2194 Johannesburg

Date of revision of the text: 2013-12

thermometer includes parts (battery, battery cover) that can be swallowed by children. Do not therefore leave the thermometer unattended when children are present. If the ambient temperature is above 35 °C immerse the thermometer tip in cold water for approximately 5 to 10 seconds before measuring the temperature.

A persistently high temperature, particularly in children, must be treated medically – please contact your doctor!

Battery replacement
The battery is empty and needs replacing when the symbol ➤ appears on the bottom right of the digital display. Remove the battery cover and replace it with a battery of the same type. Please note: the + sign on the battery must point downwards. Close the battery compartment, making sure that the catch clicks into place.

Product disposal
Please respect the environment. Thermometers and batteries should not be disposed of in household waste. Hand them into collection centres or municipal recycling centres for special waste.

Technical data
Type: Maximum thermometer
Measurement range: 32 °C to 42.9 °C
Accuracy: ±0.1 °C in the range 35.5 °C to 42 °C
±0.2 °C for the remaining LCD display, (display unit 0.1 °C)

Storage/transportation temperature: –25 °C to + 55 °C, relative humidity of ≤ 95 %

Ambient temperature during use: 10 °C to 35 °C, relative humidity of ≤ 80 %

Battery type: Alkaline-manganese battery, type LR 41, 1.55 V, battery life ~3,000 measurements

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