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Instructions for use

Product description
Covexer fast is a non-sterile permanently elastic tubular bandage for use on human skin. It is for single-use.

Indications

Covexer fast can be used

- by lay persons and healthcare professionals
- as a secondary fixation dressing for application in combination with primary dressings for the treatment of acute and chronic wounds.

under supervision of healthcare professionals

- as a moist dressing in dermatology when moistened for short-term (< 30 days) treatment in the acute phase of atopic eczema in a special moisture and ointment therapy for application in combination with liquids, ointments and creams.
- as a dry dressing in dermatology for long-term treatment in the chronic phase of atopic eczema for application in combination with externa such as ointments, creams, powder, tincture/lotions, paste,
- as an inner lining for long-term prevention of skin irritation for application in combination with compression, zinc paste, support, cast, adhesive and cohesive bandages.

Precautions

- Discontinue use and seek immediate medical advice, if skin condition worsens.
- Prior to use as moist or dry dressing in dermatology, seek advice from a healthcare professional.

• During application and removal of the tubular bandage it is essential to keep the primary dressing in place.

Application

- I. As secondary dressing to retain primary dressings**
 1. Select right size (see table with declaration of circumference). Cut off required length of Covexer fast (depending on the size of the primary dressing and length of body part).
 2. Apply Covexer fast over the end of the concerned body part.
 3. Apply Covexer fast carefully upwards over the primary dressing.
 4. In some cases an additional fixation of primary dressing might be necessary.

II. As a moist dressing when moistened (for the acute phase of atopic eczema as special moisture and ointment therapy)

1. Select right size (see table with declaration of circumference). Cut off required length of Covexer fast (depending on the size of the primary dressing and length of body part).
2. Apply the chosen externa (e.g. ointment) to the skin.
3. Immerse the bandage in lukewarm water and wring it out.
4. Apply it carefully over the affected area of the body. Avoid displacement of the applied externa.
5. In order to protect clothing, pull a second, dry layer over the damp bandage already applied.

III. As a dry dressing (during chronic phase of atopic eczema for application in combination with externa such as ointments, creams, powder, tincture/lotions, paste)

1. Select right size (see table with declaration of circumference). Cut off required length of Covexer fast (depending on the size of the primary dressing and length of body part).
2. Apply the chosen externa (e.g. ointment) to the skin.
3. Apply Covexer fast carefully upwards. Avoid displacement of the applied externa.

Incident reporting
For a patient/user/doctor/other party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices), if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Reseing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

Product disposal
In order to minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device should follow disposal procedures according to applicable local and national regulations and infection prevention standards. Dispose of the medical device with household waste.

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