Gebruichsinweise

Produktbeschreibung

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Instructions for use

Product description

Hydrosorb/Hydrosorb comfort is a transparent Hydrogel transparent consisting of an absorbent polyurethane-polyurea hybrid polymer which contains propylene glycol. The backing consists of a gas-permeable polyurethane film which is impermeable to liquids and bacteria.

In addition, Hydrosorb comfort has an adhesive border, which is coated with polyacrylate adhesive.

The transparent, write-on documentation aid facilitates complete documentation of the wound-healing process.

Properties and mode of action

Hydrosorb/Hydrosorb comfort can release moisture and absorb a limited amount of exudate. This provides a moist wound environment which has a stimulating effect on the wound healing. Hydrosorb/Hydrosorb comfort is transparent and may not stick to the wound.

Because of these characteristics, wound inspection is possible at any time without a change of dressing. Irrigation is not necessary for assessing the wound during a change of dressing.

As Hydrosorb comfort has a skin-friendly adhesive film border, no other materials are needed to secure the dressing.

Intended use/Indications

Hydrosorb/Hydrosorb comfort is suitable for the treatment of dry or lightly exuding wounds in the granulation and epithelialisation stages.

Hydrosorb/Hydrosorb comfort is intended to be applied by professional users. Wound care professionals may delegate dressing changes to patients and/or their relatives (non-professionals under supervision of professionals).

Mode of application

Select a size of Hydrosorb/Hydrosorb comfort so that the dressing overlaps the margins of the wound by approx. 1–2 cm. Surrounding skin should be dry before applying the wound dressing Hydrosorb comfort with adhesive border. Remove the blue protective foil on the side facing the wound, place Hydrosorb on the wound and secure it with additional dressing material or for Hydrosorb comfort ensure that the adhesive edges are securely attached.

To facilitate complete documentation of the wound-healing process, a transparent, write-on documentation aid is provided on the product. Note date and patient's name on the transparent documentation aid and trace the margins of the wound with an overhead marker. Then carefully roll the documentation foil off the wound dressing, remove the protective foil of the adhesive strip and stick the foil on the wound documentation form: see pictograms.

Hydrosorb: see pictograms 1a to 1h
Hydrosorb comfort: see pictograms 2a to 2h

Hydrosorb/Hydrosorb comfort can be left on wounds up to seven days. Absorption of exudate is apparent as clouding and/or blistering of the dressing. Hydrosorb/Hydrosorb comfort should be changed when the blister has approximately reached the dimensions of the wound area. To change the dressing, detach Hydrosorb comfort slightly at one edge and remove it carefully.

Possible adverse events

As any other medical device, this product can have side effects, which may not occur for everybody.

General side effects of Hydrosorb/Hydrosorb comfort:

- In rare cases Hydrosorb/Hydrosorb comfort might cause local reaction (irritation, skin reactions e.g. redness, swelling)
- In very rare cases Hydrosorb/Hydrosorb comfort might cause skin injury after removal
- In rare cases Hydrosorb/Hydrosorb comfort might cause allergic reaction cutanées après le retrait

In case you notice any effects not listed here, contact your doctor or pharmacist.

Contraindications

Do not use Hydrosorb/Hydrosorb comfort on patients who may be allergic to any of the ingredients. Do not use on wounds which are clinically infected or on 3rd degree burns.

Medical precautions

Medical assessment of the condition of the wound and the causes of the delay in wound healing are necessary before treating wounds with impaired healing. Treatment with Hydrosorb/Hydrosorb comfort cannot replace treatment of the cause of the impaired healing.

The use of Hydrosorb/Hydrosorb comfort on children under 5 years, pregnant and breastfeeding women has not been established. Caution should be undertaken by your attending physician or wound care specialists.

General instructions

Keep out of reach of children.

Date of revision of the text: 2018-08

Single use only

Do not re-sterilize

Do not use if package is damaged or open

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