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DE

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The advantages at a glance:

EN

- Hydrogel Plaster for convenient and effective treatment of superficial scalds and minor burns
- Pleasant cooling pain relief
- Accelerates the healing process
- Almost invisible
- Waterproof
- Rapid healing due to an optimal moisture balance in the wound
- The outer film is breathable and skin-friendly
- Transparent for visual inspection of the wound healing process
- Extra thin materials allow for optimal and comfortable attachment to the skin
- Doesn't stick to the wound
- Protection of the wound due to the cushioning effect of the gel
- Prevents the penetration of dirt and bacteria
- Can remain on the wound for up to 4 days

The active principle of moist wound healing:

The sterile burn plasters consist of a moist gel. They are able to release moisture and take up exudate at the same time. The innovative principle of moist wound healing actively supports the body's natural healing mechanisms and enables a quick and uncomplicated healing process.

Scientific studies have shown that skin cells in a moist wound environment reproduce more effectively and fuse together more easily, thereby promoting wound healing.

Please note:

The burn plasters must not be used on infected wounds or if you are hypersensitive to any any of its ingredients.

Please consult your physician:

- If the wound shows signs of infection (e.g. excessive redness, overheating, swelling or pain)
- If the wound does not begin to heal within 4 days
- For the treatment of large surface burns

The burn plaster may occasionally cause skin reactions in susceptible subjects.

Instructions for use:

- Always wash your hands before and after wound treatment.
- Where possible, cool the afflicted area for approx. 5 minutes under cold running water before you apply the plaster. Then dry the surroundings of the wound carefully with a clean, lint-free and, where possible, sterile material.
- Start by removing the white protective backing on one side from the underside of the plaster.
- Lay the burn plaster with the exposed side on the wound and then remove the remaining protective backing.
- Press the plaster down evenly and particularly firmly around the edges.
- Carefully remove the thin, upper protective cover.
- Carefully monitor the healing process of the burn wound.
- The burn plasters should be used until the wound has completely healed.
- To remove the plaster, lift it at one edge and carefully remove the whole plaster.
- Do not store at temperatures above 40 °C.
- Keep out of reach of children.
- Contents remain sterile unless peel packs are damaged.

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