Produktbeschreibung

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Instructions for use**Product description**

Sorbalgon is a wound dressing (or ribbon) consisting of calcium alginate fibres and polysorbate. In contact with sodium salts – as found in wound exudate – the alginate fibres become gelatinous creating a moist wound environment.

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Indications

For all moderate to heavily exuding superficial or/and cavity wounds.

Mode of application

Cleanse the wound in accordance with standard nursing practice. Apply the dressing to the wound or pack the wound with a Sorbalgon/T ribbon and secure with a suitable secondary dressing. In case of dry wounds Sorbalgon can be moistened with Ringer's solution. Sorbalgon may be left in-situ for extended periods (up to several days), depending on the level of wound secretion. The dressing should be changed once it has fully turned into a gel in accordance with nursing standards. Should any fibres remain in the wound, these may be easily removed by a wound irrigation solution* and are broken down naturally by the body.

Side effects

Allergic reactions may occur as potential side effects.

Contraindications

Do not use Sorbalgon/T on any patients who may be allergic to any of its ingredients.

Special precautions

In the absence of available data supporting the use of this dressing on sensitive population groups such as infants, children, pregnant or nursing women, and in the absence of data to the contrary, on these population groups this dressing should be used with caution following a clinician's recommendation. Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information not on request.

Medical assessment of the condition of the wound and the causes of the delayed wound healing are necessary before treating wounds with impaired healing. Treatment with Sorbalgon cannot replace treatment of the cause of the impaired healing. In the event of full thickness second degree and third degree burns, Sorbalgon may be used under medical supervision following debridement of necrotic tissue or surgical treatment.

General instructions

Store in a dry place. Contents sterile unless peel packs are damaged.

Do not resterilize. Keep out of reach of children.

* e.g. Ringer's solution or Ringer's rinsing solution without any additives

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PAUL HARTMANN AG