**HARTMANN**

# Cosmos<sup>®</sup>

Hydrocolloid plasters for blisters  
Hydrokoloidní náplasti na puchýře  
Hydrokoloidné náplasti na pluzgiera

Instructions for use

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Návod na používanie

 PAUL HARTMANN AG  
Paul-Hartmann-Str. 12  
89522 HEIDENHEIM, GERMANY  
www.hartmann.info

 Distributor  
CZ – HARTMANN-RICO a.s.  
66471 Veverská Bítýška  
SK – HARTMANN-RICO spol. s r.o.  
85101 Bratislava

069 8374 (200422)

  
0 1 2 3

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**EN Instructions for use****Product description**

Hydrocolloid plasters are a non-sterile, disposable, self-adhesive patch composed of hydrocolloids and occlusive agents (for treatment of wounds in a moist environment). This product is meant for treatment of humans by lay users.

**Composition**

The plaster is composed of two layers of adhesive hydrocolloid material (containing carboxymethyl cellulose): one acting as wound pad and the other acting as the dressing backing material. Silicone paper protects the side of the plaster with adhesive coating.

**Properties**

The main function of the product is the provision and maintenance of optimal hydration of the exposed tissues, by the moist wound healing properties of the hydrocolloid formulation, and the protection of blisters, by the adhesion properties of the backing material. The self-adhesive hydrocolloid dressing contains a layer which absorbs exudate and expands into a gel-like mass, thus providing cushioning for the treated damaged area and helping to relieve pressure and friction.

**Intended use**

Hydrocolloid plasters are non-active, single-use, self-adhesive, non-sterile devices consisting of a self-adhesive hydrocolloid wound contact layer and an adhesive coated backing material (carrier material), protected with a silicone paper, for the protection and treatment of blisters and absorption of exudates. The products are intended for humans, without restrictions (in age group, weight range, health status or condition), for use by lay users, in contact with intact skin (healthy skin) and breached skin. The products do not contain resorbable components, medicinal products/drug substances (derived from human blood or plasma or not), tissues or cells of human or animal origin, radioactive substances, nanomaterials or other hazardous components.

**Indication for use**

Protection and treatment of blisters and absorption of exudates.

**Contraindications**

The hydrocolloid plasters must not be used on non-treated infected blisters or minor injuries, major injuries, highly bleeding or exudative wounds or for pressure ulcers; or for any purpose other than those listed in the Intended purpose.

**Side effects**

Patients who are allergic or hypersensitive to any of the product ingredients may experience an adverse reaction.

**Product disposal**

The disposable components and used plasters can be disposed of normally with household waste.

**Mode of application**

- Wash hands and ensure the affected area is clean, apply the plaster only to dry and oil-free skin.
- Open the package and pull out the plaster.
- Warm the plaster with your hands before use.
- Remove the first protective paper and place the exposed part of the plaster to the affected skin area.
- Then gently pull off the second protective paper and apply the remaining part of the plaster. Gently mould the plaster edges to ensure optimum adhesion.
- Products containing hydrocolloid can be applied for a maximum of 4 days for a single product. To remove the plaster early or after the maximum application time has been reached, moisten it briefly with warm water.

**Special precautions**

- Single-use. Do not reuse.
- Apply the plaster only if the hydrocolloid pad covers the entire surface of the blister.
- Consult a physician before applying the plaster to a potentially infected blister, or if the blister shows signs of infection while using the plaster (e.g., redness, overheating, swelling, or pain).
- In case of diabetes mellitus, always consult your doctor about the use of the plaster.
- Do not puncture an unruptured blister and put yourself at risk of infection.
- Do not use the product for more than 4 days.
- Store the plasters in a cool (not refrigerated) and dry place, keep out of the reach of children.

**Incident reporting**

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

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