IC Instrucțiuni za uporaba

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EN Instructions for use

Product description

Hydrofilm is a self-adhesive, transparent film dressing made of a semi-permeable, waterproof, and transparent polyurethane film coated with a polycrylate adhesive.

Properties
The semi-permeable polyurethane film is coated with an adhesive. The film facilitates the free passage of water vapour and gases and it is waterproof. As Hydrofilm is completely transparent, the wound can be inspected without removing the dressing.

Intended use

Hydrofilm is a non-invasive, single-use, self-adhesive, transparent film dressing. It is intended to be used in a sterile condition by healthcare professionals and lay persons in humans for protection against bacterial penetration, external contaminants, and mechanical irritation of dry or slightly exuding, primary healing wounds, and for almost healed epithelising wounds. It is also intended to be used by healthcare professionals for securing wound dressings, catheters and cannulas. Hydrofilm can remain in situ for up to 5 days. However, medical assessment of the wound condition is recommended to determine when the dressing needs to be changed. A new device can be applied immediately, and the total wear time should not exceed 30 days. Additionally, healthcare professionals can use Hydrofilm in negative pressure wound therapy with Vivano System as a sealing element on intact skin, where it can remain for up to 5 days. A new device can be applied immediately, and the total wear time may exceed 30 days.

Indications
Hydrofilm can be used for protection against bacterial penetration, external contaminants, and mechanical irritation of dry or slightly exuding, primary healing wounds, for almost healed epithelising wounds as well as for the securing of wound dressings, catheters and cannulas. Additionally, it can be used as a sealing element in negative pressure wound therapy.

Contraindications
Hydrofilm must not be used as a primary dressing on clinically infected, bleeding or highly exudative wounds.

Mode of application (see pictograms I–IV)
Clean the application area thoroughly and ensure that the skin is dry and free from creams and ointments. Hairly skin areas should be shaved before applying Hydrofilm. (I) Open the peel pouch and take out the dressing. Hydrofilm is protected by 4 covers, numbered 1 to 4 in the order of application. Slightly remove the protective covers 1 and 2, then (II) place the Hydrofilm adhesive side on the wound. Fully remove the protective covers 1 and 2, gently press the entire film onto the skin, conforming it to the body contours. (III) Remove the protective cover 3. (IV) Then remove the protective cover 4 and gently press Hydrofilm to the skin.

Hydrofilm can be removed almost painlessly by overstretching the material. To do this, loosen an edge and carefully take off the dressing by overstretching it.

Special precautions

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

Side effects
Adverse reactions may occur in patients who are allergic or hypersensitive to acryly adhesives.

Incident reporting
For a patient/unsafe/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices), if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

Product disposal
In order to minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device should follow disposal procedures according to applicable local laws, rules, regulations and infection prevention standards. Dispose of the medical device with household waste.

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