60mm

60mm

60mm

60mm

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EN Instructions for use**Intended use**

Sterile, single-use, self-adhesive hydrocolloid plasters for coverage and treatment of abrasions. The products are intended for the use on human skin and can be applied by lay users.

Properties

Whilst traditional plasters work via dry wound healing, which allows the covered tissue to dehydrate and dry out, the hydrocolloid plaster is based on the mode of action of moist wound healing (occlusive properties).

Self-adhesive hydrocolloid dressings contain a layer which absorbs exudate and expands into a gel-like mass, thus providing cushioning for the treated damaged area and helping to relieve pressure and friction.

90mm

60mm

60mm

60mm

60mm

60mm

Important instructions for use

- Ensure that the affected area is clean, dry and oil-free before applying the plaster.
- Only remove the plaster when it becomes loose. To remove the plaster earlier, wet it with warm water.
- Do not use on infected or severely bleeding wounds or third-degree burns.
- Consult a doctor if the wound shows signs of infection before or during application (e.g. redness, swelling) and in case of diabetes or problems with circulation.

Application

- Wash hands and ensure the affected area is clean, dry and oil free before applying the plaster.
- Open the sealing paper and extract the plaster.
- Remove the first protective paper and apply the exposed part of the plaster to the affected skin area.
- Then gently pull the second protective paper and apply the rest of the plaster. Gently mould the plaster edges to ensure optimum adhesion.

Customer queries

If necessary, please contact us at the relevant address listed at the end of the instructions. Any serious incident (which may lead to the death, temporary or permanent serious deterioration of the health of the patient, user or other person, or a serious public health threat) caused by this product should be reported to the manufacturer and the Competent Authority of Member State in which the user or patient is established.

Date of revision and revision number of the text: 2022-02-10, Version 1

**Instructions particulières · Speciale instructies · Besondere Hinweise
Avvertenze particolari · Special instructions**

	Dispositif médical · Medisch hulpmiddel · Medizinprodukt Dispositivo medico · Medical Device
	Fabricant · Fabrikant · Hersteller · Fabricante · Manufacturer
	Date limite d'utilisation · Te gebruiken tot · Verwendbar bis Data di scadenza · Use-by date
	Date de fabrication · Productiedatum · Herstellungsdatum Data di fabbricazione · Date of manufacture
	Code de lot · Lotnummer · Fertigungslosnummer · Codice del lotto · Batch code
	Référence catalogue · Artikelnummer · Artikelnummer Numero di catalogo · Catalogue number
	Consulter les instructions d'utilisation · Raadpleeg de gebruiksaanwijzing · Gebrauchsanweisung beachten Consultare le istruzioni per l'uso · Consult instructions for use
	Stérilisé par irradiation · Gesteriliseerd met straling Strahlensterilisiert · Sterilizzato con radiazioni ionizzanti Sterilized using irradiation
	Ne pas réutiliser · Niet opnieuw gebruiken · Nicht wiederverwenden Non riutilizzare · Do not re-use
	Ne pas utiliser si l'emballage est endommagé · Niet gebruiken als de verpakking beschadigd is · Bei beschädigter Verpackung nicht verwenden Non utilizzare se l'imballaggio non è integro Do not use if package is damaged

Craint l'humidité · Droog bewaren · Trocken aufbewahren
Mantenere asciutto · Keep dry

Conserver à l'abri de la lumière du soleil · Uit het zonlicht houden · Vor Sonnenlicht schützen · Tenere lontano dalla luce
Keep away from sunlight

HARTMANN**DermaPlast®**

Pansement pour éraflures
Schaafwondenpleister
Schürfwundenpflaster
Cerotto per abrasioni
Abrasion plaster

10917/0 (05/05/22)

CMC Consumer Medical Care GmbH
Eichendorffstraße 12-14
89567 SONTHEIM/BRENZ, GERMANY
www.cmc-group.de

0 0 6 8