40mm

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EN Instructions for use

Intended use

Non-sterile, single-use, self-adhesive hydrocolloid plasters for the protection and treatment of blisters. The products are intended for the use on human skin and can be applied by lay users.

Important instructions for use

- Ensure that the affected area is clean, dry and oil-free before applying the plaster.

- Only remove the plaster when it becomes loose. To remove the plaster earlier, wet it with warm water.
- Do not use on infected or severely bleeding blisters.
- Consult a doctor if the wound shows signs of infection before or during application (e.g. redness, swelling) and in case of diabetes or problems with circulation.

Application

- Only use the plaster if the hydrocolloid pad completely covers the blister.
- Wash hands and ensure that the affected area is clean, dry and oil-free before applying the plaster.
- Remove the protective paper on the adhesive side.
- Apply the plaster to the affected skin area. Gently mould the plaster edges to ensure optimum adhesion.
- In case you want to remove the blister plaster earlier, first wet it briefly with warm water.

Customer queries

If necessary, please contact us at the relevant address listed at the end of the instructions. Any

serious incident (which may lead to the death, temporary or permanent serious deterioration of the health of the patient, user or other person, or a serious public health threat) caused by this product should be reported to the manufacturer and the Competent Authority of Member State in which the user or patient is established.

Date of revision and revision number of the text: 2022-02-10, Version 1

Instructions particulières · Speciale instructies Besondere Hinweise · Avvertenze particolari Special instructions

	Dispositif médical · Medisch hulpmiddel · Medizinprodukt Dispositivo medico · Medical Device
	Fabricant · Fabrikant · Hersteller · Fabbricante · Manufacturer
	Date limite d'utilisation · Te gebruiken tot · Verwendbar bis Data di scadenza · Use-by date

	Date de fabrication · Productiedatum Herstellungsdatum · Data di fabbricazione · Date of manufacture
	Code de lot · Lotnummer Fertigungslosnummer · Codice del lotto · Batch code
	Référence catalogue · Artikelnummer Artikelnummer · Numero di catalogo · Catalogue number
	Consulter les instructions d'utilisation · Raadpleeg de gebruiksaanwijzing Gebrauchsanweisung beachten Consultare le istruzioni per l'uso Consult instructions for use
	Ne pas réutiliser · Niet opnieuw gebruiken · Nicht wiederverwenden Non riutilizzare · Do not re-use
	Craint l'humidité · Droog bewaren Trocken aufbewahren · Mantenere asciutto · Keep dry

Conserver à l'abri de la lumière du soleil · Uit het zonlicht houden
Vor Sonnenlicht schützen · Tenere lontano dalla luce · Keep away from sunlight

0 0 6 8
CMC Consumer Medical Care GmbH
Eichendorffstraße 12-14
89567 SONTHEIM/BRENTZ, GERMANY
www.cmc-group.de

849 3653/0 (120522)

DermaPlast®

Pansements pour ampoules
Blarenpleisters
Blasenpflaster
Cerotti per vesciche
Blister plasters