60mm

60mm

60mm

60mm

60mm

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EN Instructions for use**Intended use**

Sterile, single-use, self-adhesive hydrogel plasters for the protection and treatment of burns. The products are intended for the use on human skin and can be applied by lay users.

The principle of moist wound healing

Sterile burn plasters consist of a moist gel. They are able to release moisture and take up exudate at the same time. The innovative principle of moist wound healing actively supports the body's natural healing mechanisms and enables a quick and uncomplicated healing process.

Important instructions for use

- Ensure that the affected area is clean, dry and oil-free before applying the plaster.
- Do not use on infected or highly exuding wounds or third-degree burns.
- The dressing should not be used in case of sensitivity to propylene glycol and acrylic adhesive.
- In case of reddening or allergy, please seek medical attention.

Application

1. Remove liner (1 and 2).
2. Stick to wound.
3. Remove application aid (3).
4. Check healing process regularly and renew plaster.

**Customer queries**

If necessary, please contact us at the relevant address listed at the end of the instructions. Any serious incident (which may lead to the death, temporary or permanent serious deterioration of the health of the patient, user or other person, or a serious public health threat) caused by this product should be reported to the manufacturer and the Competent Authority of Member State in which the user or patient is established.

Date of revision and revision number of the text: 2022-02-10, Version 1

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CMC Consumer Medical Care GmbH