60mm

60mm

60mm

60mm

60mm

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**EN Instructions for use****Intended use**

Sterile, single-use, self-adhesive hydrogel plasters for the protection and treatment of burns. The products are intended for the use on human skin and can be applied by lay users.

**The principle of moist wound healing**

Sterile burn plasters consist of a moist gel. They are able to release moisture and take up exudate at the same time. The innovative principle of moist wound

60mm

60mm

60mm

60mm

60mm


healing actively supports the body's natural healing mechanisms and enables a quick and uncomplicated healing process.

#### Important instructions for use

- Ensure that the affected area is clean, dry and oil-free before applying the plaster.
- Do not use on infected or highly exuding wounds or third-degree burns.
- The dressing should not be used in case of sensitivity to propylene glycol and acrylic adhesive.
- In case of reddening or allergy, please seek medical attention.

#### Application

1. Remove liner (1 and 2).
2. Stick to wound.
3. Remove application aid (3).
4. Check healing process regularly and renew plaster.

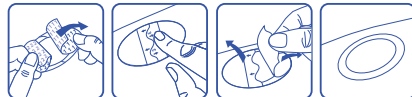#### Customer queries

If necessary, please contact us at the relevant address listed at the end of the instructions. Any serious incident (which may lead to the death, temporary or permanent serious deterioration of the health of the patient, user or other person, or a serious public health threat) caused by this product should be reported to the manufacturer and the Competent Authority of Member State in which the user or patient is established.

Date of revision and revision number of the text: 2022-02-10, Version 1

#### Instructions particulières · Speciale instructies · Besondere Hinweise Avvertenze particolari · Special instruction

	Dispositif médical · Medisch hulpmiddel · Medizinprodukt Dispositivo medico · Medical Device
	Fabricant · Fabrikant · Hersteller · Fabricante · Manufacturer
	Date limite d'utilisation · Te gebruiken tot · Verwendbar bis Data di scadenza · Use-by date
	Date de fabrication · Productiedatum · Herstellungsdatum Data di fabbricazione · Date of manufacture
	Code de lot · Lotnummer · Fertigungslosnummer · Codice del lotto · Batch code
	Référence catalogue · Artikelnummer · Artikelnummer Numero di catalogo · Catalogue number
	Consulter les instructions d'utilisation · Raadpleeg de gebruiksaanwijzing · Gebrauchsanweisung beachten Consultare le istruzioni per l'uso · Consult instructions for use
	Stérilisé par irradiation · Gesteriliseerd met straling Strahlensterilisiert · Sterilizzato con radiazioni ionizzanti Sterilized using irradiation
	Ne pas réutiliser · Niet opnieuw gebruiken · Nicht wiederverwenden Non riutilizzare · Do not re-use
	Ne pas utiliser si l'emballage est endommagé · Niet gebruiken als de verpakking beschadigd is · Bei beschädigter Verpackung nicht verwenden Non utilizzare se l'imballaggio non è integro Do not use if package is damaged

	Craint l'humidité · Droog bewaren · Trocken aufbewahren Mantenere asciutto · Keep dry
	Conserver à l'abri de la lumière du soleil · Uit het zonlicht houden · Vor Sonnenlicht schützen · Tenere lontano dalla luce Keep away from sunlight

HARTMANN

# DermaPlast®

Pansement pour brûlures  
Brandwundenpleister  
Brandwundenpflaster  
Cerotto per ustioni  
Burn plaster

10915/0 (240822)

 CMC Consumer Medical Care GmbH  
Eichendorffstraße 12-14  
89567 SONTHEIM/BRENZ, GERMANY  
www.cmc-group.de

0 0 6 8