Complete Liner

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1 Paar / 1 Pair

sigvaris

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INSTRUCTIONS FOR USE AND CARE

Product Description

- Provides 10–15 mmHg of compression in the foot, ankle and calf area
- Soft liner with anti-bacterial wash and moisture-regulating and odor-control properties

Intended Purpose

The Complete Liner protects the skin and ensures comfort when worn underneath SIGVARIS GROUP inelastic wraps. It also provides gentle compression in the foot, ankle and calf area, thereby helping to reduce edema when used with inelastic wraps from SIGVARIS GROUP. The product is intended for use on the (lower) legs. This product is intended for multiple use and is non-sterile. For further information on indications, warnings, contraindications and target groups, please refer to the instructions for use for the relevant wraps.

Content

1 pair of Complete Liners with compression in the foot, ankle and calf area

Material

66% cotton
19% polyamide
9% polyester
6% elastane

Care

- Please wash before first use
- Wash on gentle cycle using a mild detergent
- DO NOT use bleach or fabric softeners
- Dry on low/no heat or air dry
- Wash liners after every use

Warnings and Precautions

Only for use with inelastic wraps from SIGVARIS GROUP. Please read these instructions carefully and consult your physician if you have any health concerns. Your SIGVARIS GROUP compression garments should be comfortable and never hurt.

Use

While the Complete Liners are suitable for day and nighttime wear, the garment should be removed once a day for skin hygiene purposes and to allow any wound care to be carried out. The liners should then be put back on afterwards. Frequency of garment removal for skin checks, hygiene, and/or wound care may vary according to each individual patient's needs and the prescribing provider's recommendations.

Disposal

There are no special requirements for disposing of the medical compression product. This can be disposed of as residual or household waste. Please comply with local regulations for the disposal of textiles.

..... **Made without natural rubber latex**

Please note: consult your physician. The precautions and contraindications listed here are guidelines only and are no substitute for medical advice.

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SIGVARIS and SIGVARIS GRUPP, are registered trademarks of SIGVARIS AG, CH-9014, St. Gallen/Switzerland, in many countries worldwide. © 2023 Copyright by SIGVARIS AG, CH-9014 St.Gallen/Switzerland