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EN Instructions for use
Product description
HydroTAc Transparent™ is a transparent hydrogel dressing consisting of hydrated polyurethane and glycerine. The backing consists of a water vapour permeable polyurethane film which is impermeable to liquids and bacteria.

In addition, HydroTAc Transparent Comfort has an adhesive border, which is coated with an acrylic based adhesive.

Properties and mode of action
HydroTAc Transparent can release moisture and absorb a limited amount of exudate. This provides a moist wound environment which has a stimulating effect on wound healing.

HydroTAc Transparent provides and provides an atraumatic dressing change. Because of the transparency, wound inspection is possible at any time without having to remove the dressing.

As HydroTAc Transparent Comfort has a skin-friendly adhesive film border, no other materials are needed to secure the dressing.

Intended use / Indications

HydroTAc Transparent™ is suitable for the treatment of dry or lightly exuding wounds in the granulation and epithelialization stages. HydroTAc Transparent is also suitable for the application on the outer eye region to protect the eyes during general anaesthesia.

HydroTAc Transparent is intended to be applied by healthcare professionals.

These may delegate dressing changes to patients and/or their relatives (instructed lay persons).

Mode of application
Select a size of HydroTAc Transparent so that the wound pad overlaps the margins of the wound by approx. 1 – 2 cm. If required, the product HydroTAc Transparent can be cut to desired size, using sterile scissors only. Surrounding skin should be dry before applying HydroTAc Transparent Comfort with adhesive border.

HydroTAc Transparent: Remove the blue protective layer following the numbering (1 & 2). Place the dressing with the gel-impregnated side on the wound and fix with suitable sterile dressing material.

HydroTAc Transparent Comfort: Remove the blue protective layer following the numbering (1 & 2), place the dressing with the gel-impregnated side on the wound and press down the adhesive edges firmly. Remove the protective top layer following the numbering (3 & 4).

HydroTAc Transparent can be left on wounds up to seven days. To change the dressing, detach HydroTAc Transparent slightly at one edge and remove it carefully.

Product disposal
To minimize the risk of potential infection hazards or environmental pollution, disposable components of HydroTAc Transparent should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

Contraindications
Do not use HydroTAc Transparent on patients with an intolerance to any of its ingredients, especially polyurethane, glycerin or acrylic based adhesive. Do not use on wounds which are clinically infected or on 3rd degree burns.

Possible adverse events
As any other medical device, this product can have side effects, which do not necessarily occur for everybody.

General side effects of HydroTAc Transparent:

- In rare cases HydroTAc Transparent might cause local reactions (irritation, skin reactions e.g. redness, swelling)
- In rare cases HydroTAc Transparent might cause allergic reactions

In case you notice any effects not listed here, contact your doctor or pharmacist.

Special precautions
Medical assessment of the condition of the wound and the causes of any delay in wound healing are necessary before treating wounds with impaired healing. Treatment with HydroTAc Transparent cannot replace treatment of the cause of impaired healing. Only application on the outer eye area when the eyelid is closed.

Reusing a single-use medical device is dangerous. Reprocessing devices, or use to reuse them, may seriously damage their integrity and their performance.

Before opening the product packaging, inspect it carefully for signs of damage. Do not use the product if there are visible signs of damage, such as cracks, channels, pinholes, or nonuniform, torn or incomplete seals.

Incident reporting
For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation (EU) 2017/745 on Medical Devices), if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorized representative and to your national authority.

General instructions
The product may discolour over time without impact on the product performance. Keep out of reach of children.

*Any reference to HydroTAc Transparent contained in the text refers to all presentations, including Comfort versions; unless otherwise specified.

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PT – PAUL HARTMANN S.p.A. - 37139 Verona