

DermaPlast® / Cosmos® ACTIVE Kinesiology Tape

General Product Description/Intended Purpose

DermaPlast® / Cosmos® ACTIVE Kinesiology Tapes are elastic adhesive tapes used for taping muscles, joints and ligaments in physiotherapy, sport traumatology, pain therapy and other treatments where no complete immobilization of the taped area is aimed for.

The special cotton structure leads to high stretching properties. The Kinesiology Tape with its breathable acrylate adhesive is latex-free, can be worn at any time and is very comfortable to wear. The Kinesiology Tape includes a detailed taping instructions and is available in 3 different colors, blue, pink and skin-colored.

Application/Indication

DermaPlast® / Cosmos® ACTIVE Kinesiology Tape is a breathable, air-permeable and waterproof tape for sport, therapy and leisure applied on intact skin.

Reference Numbers

DermaPlast® / Cosmos® ACTIVE Kinesiology Tape is available in 3 colour variations and 3 languages circles:

Article-No.:	Product name:	Version:	Languages:	Packaging:
522040	DermaPlast® ACTIVE Kinesiology Tape	Blue	DE, FR, IT, NL	P1, OK8
522041	DermaPlast® ACTIVE Kinesiology Tape	Pink	DE, FR, IT, NL	P1, OK8
522042	DermaPlast® ACTIVE Kinesiology Tape	Beige	DE, FR, IT, NL	P1, OK8
522140	DermaPlast® ACTIVE Kinesiology Tape	Blue	ES, PT	P1, OK8
522141	DermaPlast® ACTIVE Kinesiology Tape	Pink	ES, PT	P1, OK8
522142	DermaPlast® ACTIVE Kinesiology Tape	Beige	ES, PT	P1, OK8
522240	Cosmos® ACTIVE Kinesiology Tape	Blue	CZ, SK	P1, OK8
522241	Cosmos® ACTIVE Kinesiology Tape	Pink	CZ, SK	P1, OK8
522242	Cosmos® ACTIVE Kinesiology Tape	Beige	CZ, SK	P1, OK8

Warnings

- Pregnant women should visit a doctor for advice before taping.
- Do not expose the tape to direct sunlight or high temperatures.
- Keep the product out of reach of children.
- When applying the tape, make sure it does not pinch the skin. Apply the tape loosely!
- Should applying the tape cause prolonged skin irritation in the form of itching, redness or rash for over 30 minutes, the tape should be removed immediately in accordance with the instructions. Contact a doctor if necessary.
- Do not tape over fresh injuries or thromboses; and do not apply in cases of skin allergies or skin irregularities.
- Do not use the tape in connection or reciprocally with other treatment methods (fango, massage therapies, electrical stimulation treatment).
- Before applying, the skin must be clean and free of grease.
- If you are not sure how to apply the tape correctly, you can contact a therapist for an initial professional therapeutic treatment.
- It is advisable not to wear the tape for longer than 7-10 days, and then to remove it in order to protect the skin. Application can be repeated after 3-5 days for several times. Any remaining adhesive residue must be removed.

DermaPlast[®] / Cosmos[®] ACTIVE Kinesiology Tape

- In the event of acute injuries or complex symptoms you should consult a doctor or physiotherapist regarding taping.
- Careful use in patients with a history of allergy of glue and sensitivity to adhesive allergy. If in doubt, ask your doctor.
- It is advisable to consume liquid frequently while you are taped.
- Avoid heat build-up due to clothing (wool/cotton)

Single Use Device

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

Product Disposal

To minimize the risk of potential infection hazards, or environmental pollution, disposable components of the product should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

Packaging Materials

All packing materials and packing components conform to the relevant essential requirements (specific to manufacturing and composition, reusable nature and recoverable nature of packing) of the Packaging and Packaging Waste Directive 94/62/EC and amended directive 2004/12/EC respectively.

Incident Reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority.

Material Characteristics

Kinesiology Tape (5m x 5cm)

Product component:	Material composition
Elastic tape material:	95% Cotton und 5% Spandex (Elastane)
Adhesive coating:	pressure sensitive acrylic adhesive, breathable
Release paper:	coated paper

DermaPlast® / Cosmos® ACTIVE Kinesiology Tape

Product Characteristics

Kinesiology Tape (5m x 5cm)	
Property	Characteristics
Construction and dimensions:	5m x 5cm (LxW) per spool;
Printing/colour:	no printing / blue or pink or beige
Elongation:	180±30 %
Adhesive coating:	transparent
Coat weight:	45-55 g/m ²
Peel adhesion	3-12 N/25mm
Storage/Transport conditions:	no storage permanently above 30°C
Shelf life:	3 years
(Technical modification subject to change)	

Labelling

LOT number with 5-digit code:

e.g.: V2022



V

2022

letter

consecutive number

Manufacturing Date with 8-digit code:

e.g.: 2020-06-01



2021

-

06

-

01

year

month

day

Use-by-Date with 8-digit code:

e.g.: 2022-06-01



2023

-

06

-

01

year

month

day

Medical Device



DermaPlast® / Cosmos® ACTIVE Kinesiology Tape is a class I medical device in accordance with the Regulation (EU) 2017/745 on Medical Devices (MDR) and bears the CE-mark.

UMDNS No.: 10-289

DermaPlast® / Cosmos® ACTIVE Kinesiology Tape

CND Code: M0304020301 ELASTIC SUPPORT BANDAGES, ADHESIVE, LONG EXTENSIBILITY (>140%)

Unique Device Identifier (UDI)



Each product and all higher levels of packaging shall be labeled with an UDI code (implementation date according to Regulation 2017/745/EU on Medical Devices)

Legal Manufacturer



MEDINEO GmbH
Neumühle 2a
97727 Fuchsstadt, Germany

Distributor



DE – PAUL HARTMANN AG, 89522 Heidenheim
CH – IVF HARTMANN AG, 8212 Neuhausen
AT – PAUL HARTMANN Ges.m.b.H., 2355 Wiener Neudorf
IT – Paul Hartmann S.p.A., 37139 Verona
FR – Lab. PAUL HARTMANN S.à.r.l Châtenois 67607 Sélestat CEDEX
BE – N.V. PAUL HARTMANN S.A. 1480 Saintes Sint-Renelde
NL – Paul Hartmann B.V. 6546 BC Nijmegen
ES – Laboratorios HARTMANN S.A. 08302 Mataró
PT – PAUL HARTMANN LDA 2685-378 Prior Velho
CZ – HARTMANN-RICO a.s., 66471 Veverská Bítýška
SK – HARTMANN-RICO spol. s r.o., 85101 Bratislava
RO – SC PAUL HARTMANN SRL 020 194 București

Latest Date of Revision: 2020-09-02