

MoliCare® Premium Elastic , incontinence slip

General Product Description/Intended Purpose

MoliCare® Premium Elastic are Slip products with elasticated side panels and attached re-closable tapes used for incontinence care. The products are suitable for usage in case of bladder weakness, urinary and faecal incontinence with several severities. The products are intended for bedridden as well as mobile patients/residents (for humans). The products are for single use only and should be regularly changed as needed. MoliCare® Premium Elastic can be used in home care environment and professional healthcare facilities. The application can be performed by lay person and/or health care professional.

Application/Indication

Several severities of urinary and faecal incontinence



MoliCare® Premium Elastic , incontinence slip

Reference Numbers*

Packaging	Product Size	S	M	L	XL
	Hip-/waist girth (cm)	70-90	85-120	115-145	140-175
P30	5 drops	165 171 165 181	165 172 165 182	165 173 165 183	/
	6 drops	166 271 165 281	166 272 165 282	166 273 165 283	/
	7 drops	167 371 165 381	167 372 165 382	167 373 165 383	/
	8 drops	/	/	/	/
	9 drops	/	/	/	/
	10 drops	/	/	/	/
	P26	5 drops	/	/	/
6 drops		/	/	/	/
7 drops		/	/	/	/
8 drops		165 471 165 481	165 472 165 482	/	/
9 drops		165 571 165 581	165 572 165 582	/	/
10 drops		/	/	/	/
P24		5 drops	/	/	/
	6 drops	/	/	/	/
	7 drops	/	/	/	/
	8 drops	/	/	165 473 165 483	/
	9 drops	/	/	165 573 165 583	/
	10 drops	/	/	/	/
	P22	5 drops	/	/	/
6 drops		/	/	/	/
7 drops		/	/	/	/
8 drops		/	/	/	/
9 drops		/	/	/	/
10 drops		165 671 165 681	/	/	/
P14	5 drops	/	/	/	/
	6 drops	/	/	/	165 274 165 284
	7 drops	/	/	/	165 374 165 384
	8 drops	/	/	/	165 474 165 484
	9 drops	/	/	/	165 574 165 584
	10 drops	/	165 672 165 682	165 673 165 683	165 674 165 684
	P2	5 drops	165 191	165 192	165 193
6 drops		165 291	165 292	165 293	165 294
7 drops		165 391	165 392	165 393	165 394
8 drops		165 491	165 492	165 493	165 494
9 drops		165 591	165 592	165 593	165 594
10 drops		165 691	165 692	165 693	165 694

*the full assortment may not be available in all countries

MoliCare® Premium Elastic , incontinence slip

Single Use Device

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request. Reuse of the product could lead to microbiological contamination and/or mild infections of the skin.

Product Disposal

To minimize the risk of potential infection hazards, or environmental pollution, disposable components of MoliCare® Premium Elastic should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

Incident Reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority.

Product Performance Characteristics

Total Weight (g)	S	M	L	XL
5 drops	73,5	85,2	101,6	/
6 drops	78,0	94,7	108,0	129,9
7 drops	86,0	103,6	114,3	146,5
8 drops	93,1	126,2	134,8	161,1
9 drops	101,0	130,0	151,5	177,9
10 drops	113,6	168,9	195,2	195,4

* $\leq \pm 10\%$ tolerance

MoliCare® Premium Elastic , incontinence slip

Material Characteristics

Part	Material
Back sheet:	Polyethylene, Nonwoven-film-laminate, White
Absorbent core (3-layers):	1. layer: Kraft Pulp* 2. layer: Kraft Pulp*- with Superabsorbent Polymer 3. layer: Kraft Pulp, „Curled Fiber“ *Bleached Elementary Chlorine Free
Adhesive:	Hotmelt Adhesive
Top sheet:	Polypropylene - Nonwoven, Hydrophilic, White
Cuff:	Polypropylene - Nonwoven, Hydrophobic, White
Elastics:	Elastics for leg gather and standing cuffs
Elastic Side Panels with Tapes	
Nonwoven:	Polypropylene - Nonwoven, Hydrophobic, White
Elastic Material:	Polypropylene - Nonwoven, Hydrophobic (Elastic), White
Tape:	Polypropylene - Nonwoven Hook and Loop Tape
Wetness indicator:	Water Soluble Ink

Product Characteristics

Characteristics	Unit	Size S 5 - 10 Drops	Size M 5 - 9 Drops	Size M 10 Drops	Size L 5 - 9 Drops	Size L 10 Drops	Size XL 6 - 10 Drops
Total Length:*	[mm]	720	810	810	925	925	980
Total Width Abdominal Side:*	[mm]	280	320	440	440	480	480
Total Width Back Side*	[mm]	290	330	450	450	490	490
Total Width Crotch:*	[mm]	280	320	290	290	290	320
Absorbent Core Length:*	[mm]	630	660	650	780	780	780
Absorbent Core Width Abdominal Side:*	[mm]	200	200	300	200	360	240
Absorbent Core width Back Side:*	[mm]	220	245	380	265	425	350
Absorbent Core Width Crotch:*	[mm]	170	170	165	170	160	180

* $\leq \pm 10\%$ tolerance

MoliCare® Premium Elastic , incontinence slip

Labelling

Lot-No. with 8-Digit Code

e.g.: 900102822



9
year 201X

001
internal key

02
internal key

82
internal key

2
cumulated
check sum

Manufacturing date

e.g.:  2019
year

08
month

07
day

Use-by-date

e.g.:  2024
year

08
month

01
day

Shelf Life: 5 years

Medical Device



Keep Dry



Do Not Reuse



CE Marking



Latest Date of Revision: 2020-03-18