

# MoliCare® Premium lady pants

## General Product Description/Intended Purpose

### Intended Purpose:

MoliCare® Premium lady pants are pull-up pants for incontinence care for women.


The products are suitable for usage in case of bladder weakness with several severities, urinary and fecal incontinence with several severities and for usage of toilet training.

The products are intended for mobile to semi-mobile patients and restless or dement patients. The products are for single use only and should be regular changed as needed.

The products can be used in home care environment and professional healthcare facilities. The application can be performed by lay person and/or health care professional.

## Application/Indication

MoliCare® Premium lady pants are suitable for usage in case of bladder weakness with several severities, urinary and fecal incontinence with several severities and for usage of toilet training.

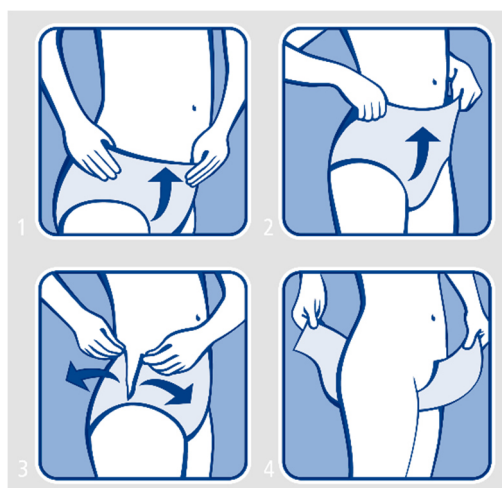

## Reference Numbers\*

Packaging		M	L
	Hip-/waist girth (cm)	80–120	100–150
P8	5 drops / green	915867	-
	7 drops / blue	915877	-
P7	5 drops / green	-	915868
	7 drops / blue	-	915870
P1	5 drops / green	915810	915811
	7 drops / blue	915815	915816

\*the full assortment may not be available in all countries

## Single use Device

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request. Reuse of the product could lead to microbiological contamination and/or mild infections of the skin.

# MoliCare® Premium lady pants

## Product disposal

To minimize the risk of potential infection hazards, or environmental pollution, disposable components of MoliCare® Premium lady pants should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

## Incident Reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority.

## Product performance characteristics

MoliCare® Premium lady pants 5 drops / green:

Characteristics	unit	M	L
total weight:*	[g]	44	51

\* $\leq \pm 10\%$  tolerance

MoliCare® Premium lady pants 7 drops / blue:

Characteristics	unit	M	L
total weight:*	[g]	67	71

\* $\leq \pm 10\%$  tolerance

## Material characteristics

Part	Material
chassis	polypropylene non-woven, hydrophobic, breathable, white with print
back sheet:	polyethylene film with polypropylene nonwoven layer, white, breathable
absorbent core (3-layer):	1. layer: kraft pulp* 2. layer: kraft pulp* - with superabsorbent polymer 3. layer: kraft pulp*, „Curled Fiber“ *bleached elementary chlorine free
adhesive:	hotmelt adhesive
top sheet:	polypropylene - nonwoven, hydrophilic, white, treated with aloe vera
cuff:	polypropylene - nonwoven, hydrophobic, white
elastics:	polyurethane based elastics for leg gathers, hip area, waist and (7 drops only) standing cuffs
product information:	printing on inside of waistband (rear side)

## Product characteristics


Dimensions	5 drops		7 drops	
	M	L	M	L
total length:	740	780	730	830
total width:	660	810	660	810
total width crotch:	165	165	210	215
absorbent core length:	340	370	490	580
absorbent core width:	120	120	180	180
absorbent core width crotch:	90	90	140	140

\* $\leq \pm 10\%$  tolerance


# MoliCare® Premium lady pants

## Labelling


### 9-digit LOT-code

e.g.:  9 001 02 82 2  
 year 201X internal key internal key internal key cumulated  
 check sum

### Manufacturing date

e.g.:  2019 08 07  
 year month day

### Expiry date

e.g.:  2024 08 01  
 year month day

Shelf life: 5 years

### Medical Device



### Keep dry



### Do not reuse



### CE-marking



**Latest date of revision:** 2019-10-04