

Idealast-haft color®

General Product Description/Intended Purpose

Idealast-haft color bandages are non-sterile medical devices that are used on intact skin in clinical and home environment for the fixation of primary and secondary dressings, splints and casts (and similar medical devices) particularly on joints. Furthermore, they are used for the stabilization of muscles, tendons, ligaments and joints with low immobilization effect as well as for slight compression in case of diseases or injuries of the musculoskeletal system (such as sprains and strains) or hematoma. Idealast-haft work especially well on conical and round parts of the body in medical interventions when acute and chronic wounds need to be treated up to long-term or when slight immobilization is needed in combination with medical devices. Idealast-haft color is a wide woven bandage and is manufactured from 93% cotton, 5% polyamide and polyurethane. The non-active devices are available in different widths and lengths and colors. They are cohesive and have medium extensibility.

Application/Indication

Idealast-haft are non-sterile medical devices that are used on intact skin in clinical and home environment for the fixation of primary and secondary dressings, splints and casts (and similar medical devices) particularly on joints. Furthermore, they are used for the stabilization of muscles, tendons, ligaments and joints with low immobilization effect as well as for slight compression in case of diseases or injuries of the musculoskeletal system (such as sprains and strains) or hematoma. Idealast-haft work especially well on conical and round parts of the body in medical interventions when acute and chronic wounds need to be treated up to long-term or when slight immobilization is needed in combination with medical devices.

Reference Numbers

REF	name	size	bandages per folding box	individually wrapped
931103	Idealast-haft color	10cmx4m	10	No
931102	Idealast-haft color	8cmx4m	10	No
931101	Idealast-haft color	6cmx4m	10	No
931100	Idealast-haft color	4cmx4m	10	No
931098	Idealast-haft color	10cmx4m	1	No
931097	Idealast-haft color	8cmx4m	1	No
931096	Idealast-haft color	6cmx4m	1	No
931095	Idealast-haft color	4cmx4m	1	No
931093	Idealast-haft color	10cmx4m	1	No
931092	Idealast-haft color	8cmx4m	1	No
931091	Idealast-haft color	6cmx4m	1	No
931090	Idealast-haft color	4cmx4m	1	No

Idealast-haft color[®]

Residual Risks, Contra-Indications and Undesirable Side Effects, Warnings

Not known.

Single Use Device

Reusing a single-use medical device is dangerous. Reprocessing devices, in order to reuse them, may seriously damage their integrity and their performance. Information available on request.

Product Disposal

In order to minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device should follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

Incident Reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority.

Idealast-haft color[®]

Product Performance Characteristics

property	value
elongation	90 %
length stretched	400 cm
width	4 cm; 6 cm; 8 cm; 10 cm
area weight stretched	approx. 132 g/m ²
Cohesive force	45 cN/cm
Unwinding force	<125 cN/cm

Labelling

Lot-No. with 9-digit code

e.g.:

	9	999	01	00	1
explanation	[9 = 2019]	[999-000]			
definition	year	serial production order			for internal purposes only

Date of Manufacturing

e.g.:		2019	04	01
		year	month	day

Use-by Date

e.g.:		2022	04	01
		year	month	day

Shelf life: 3 years

Medical Device



Unique Device Identifier (UDI)



Latest Date of Revision: 2020-05-25