

Idealflex®

General Product Description/Intended Purpose

Idealflex bandages are non-sterile medical devices that are used on intact skin in clinical and home environment for the fixation of primary and secondary dressings, splints and casts (and similar medical devices) particularly on joints. Furthermore, they are used for the stabilization of muscles, tendons, ligaments and joints with low immobilization effect as well as for slight compression in case of diseases or injuries of the musculoskeletal system (such as sprains and strains) or hematoma. This bandages work especially well on conical and round parts of the body in medical interventions when acute and chronic wounds need to be treated up to long-term or when slight immobilization is needed in combination with medical devices. Idealflex is a narrow woven bandage and is manufactured from 42% cotton and 45% polyesther and viscose. The non-active devices are available in different widths and lengths. Idealflex is an elastic compression bandage, non-adhesive and has medium extensibility. This bandage is reusable.

Application/Indication

Idealflex bandages are non-sterile medical devices that are used on intact skin in clinical and home environment for the fixation of primary and secondary dressings, splints and casts (and similar medical devices) particularly on joints. Furthermore, they are used for the stabilization of muscles, tendons, ligaments and joints with low immobilization effect as well as for slight compression in case of diseases or injuries of the musculoskeletal system (such as sprains and strains) or hematoma. This bandages work especially well on conical and round parts of the body in medical interventions when acute and chronic wounds need to be treated up to long-term or when slight immobilization is needed in combination with medical devices.

Reference Numbers

REF	name	size	bandages per folding box	individually wrapped
931283	Idealflex Bandage	6cmx5m	10	No
931284	Idealflex Bandage	8cmx5m	10	No
931285	Idealflex Bandage	10cmx5m	10	No
931286	Idealflex Bandage	12cmx5m	10	No
931287	Idealflex Bandage	15cmx5m	10	No
931288	Idealflex Bandage	20cmx5m	10	No
931290	Idealflex Bandage	6cmx5m	10	Yes
931291	Idealflex Bandage	8cmx5m	10	Yes
931292	Idealflex Bandage	10cmx5m	10	Yes
931293	Idealflex Bandage	12cmx5m	10	Yes
931294	Idealflex Bandage	15cmx5m	10	Yes
931295	Idealflex Bandage	20cmx5m	10	Yes

Idealflex[®]

Residual Risks, Contra-Indications and Undesirable Side Effects, Warnings

Not known.

Product Disposal

In order to minimize the risk of potential infection hazards, or environmental pollution, disposable components of the medical device follow disposal procedures according to applicable and local laws, rules, regulations and infection prevention standards.

Incident Reporting

For a patient/user/third party in the European Union and in countries with identical regulatory regime (Regulation 2017/745/EU on Medical Devices); if, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and/or its authorised representative and to your national authority.

Idealflex®

Product Performance Characteristics

property	value
elongation	80 %
length stretched	500 cm
width	6 cm; 8 cm; 10 cm; 12cm, 15 cm, 20 cm
area weight stretched	approx. 85 g/m ²

Labelling

	9	999	01	00	1
explanation definition	[9 = 2019] year	[999-000] serial production order			for internal purposes only

Date of Manufacture

e.g.:		2019 year	04 month	01 day
-------	--	--------------	-------------	-----------

Use-by-Date

e.g.:		2024 year	04 month	01 day
-------	---	--------------	-------------	-----------

Shelf Life: 5 years

Medical Device



Unique Device Identifier (UDI)



Latest Date of Revision: 2020-05-25